

T-COR 8™-IVD

Quick Start Guide

PLM-0594-1

Date of Issue: 2023/03

Catalog #: TC-5015-001 (Four Channel), TC-5028-001 (Five Channel), TC-5018-001 (Magnetic Mixer)

Table of Contents

- HOW TO USE THIS GUIDE..... 4**
 - PURPOSE OF THIS GUIDE.....4
 - TEXT CONVENTIONS4
 - RELATED DOCUMENTATION4
 - CONFORMING SYMBOLS.....5
 - HOW TO OBTAIN MORE INFORMATION5
 - CONTACT INFORMATION5
- IMPORTANT SAFETY INSTRUCTIONS 6**
- APPLICABLE SOFTWARE VERSIONS 6**
- UNPACKING AND ACCESSORIES 7**
 - REQUIRED CONSUMABLES (NOT SUPPLIED)8
- SET-UP REQUIREMENTS 9**
 - ENVIRONMENTAL CONDITIONS9
 - CONNECTING TO ELECTRICAL OUTLET9
 - POWERING THE DEVICE.....10
 - MAGNETIC MIXER ACCESSORY11
 - Requires the T-COR 8 Magnetic Mixer accessory, which must be purchased separately.....11*
 - Connecting to the T-COR 811*
- DEVICE UTILIZATION 12**
 - OVERVIEW OF DEVICE UTILIZATION12
 - PRE-INSTALLED USER ACCOUNTS12
 - STAND ALONE MODE13
 - DIRECT CONNECTION MODE.....13
 - ACCESS POINT CONNECTION MODE15
 - NETWORK CONNECTION MODE.....16
- A QUICK GUIDE TO RUN SETUP 17**
 - CREATING A NEW RUN17
- INDEX 20**

How to Use This Guide

Purpose of This Guide

The *T-COR 8 Quick Start Guide* provides basic information needed to operate the T-COR 8. This guide is intended for novice and experienced T-COR 8 users who will be using the device for research and human in-vitro diagnostics. This guide assumes that the user:

- Understands general techniques for preparing and handling DNA/RNA samples.
- Is familiar using Google Chrome™ web browser.
- Has a general understanding of hard drives and data storage, file transfers, and copying and pasting.



Before operating the T-COR 8, review the information in the *Tetracore T-COR 8 Hardware Manual* (PLM-0595) and the *Tetracore T-COR 8 Software Manual* (PLM-0593).

Text Conventions

This guide uses the following conventions:

- **Bold** text indicates a label. For example, a page title or text box.
- **[Bracketed]** text indicates a button. For example: Press the **[Start Run]** button.
- Underlined text indicates emphasized importance. For example: To avoid loss of data, never turn off the T-COR 8 until all runs have completed.
- *Italicized* text indicates a reference to further information. For example: See *Physical Safety Hazard*.








Related Documentation

Hard copy versions of the following related documents are included with the device. Portable document format (PDF) versions of the following related documents are available upon request.

Document	Part Number	Description
T-COR 8-IVD Software Manual	PLM-0593	The <i>Tetracore T-COR 8 Software Manual</i> provides the information needed to operate the T-COR 8.
T-COR 8-IVD Hardware Manual	PLM-0595	The <i>Tetracore T-COR 8 Hardware Manual</i> provides the information needed to utilize the T-COR 8.
T-COR 8-IVD LIS Connectivity	PLM-0573	The <i>Tetracore T-COR 8 LIS Connectivity Specification</i> documents the HL7 message sent from the T-COR 8™ device

Conforming Symbols

The following table describes symbols that may be used on the T-COR 8 device, on its packaging or in the associated documentation.

Symbol	Description
	Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the need for the user to consult the instructions for use.
	Indicates a medical device that needs to be protected from moisture.
	Indicates the temperature limits to which the medical device can be safely exposed.
	Indicates a medical device that is intended to be used as an in vitro diagnostic medical device. The T-COR 8 is a real-time thermocycler that is intended for use with PCR, RT-PCR, and Isothermal in vitro diagnostic tests. The T-COR 8 measures real-time nucleic acid amplification signals from DNA or reverse transcribed RNA and provides a qualitative result. The T-COR 8 is intended for use by laboratory professionals trained on the use of the instrument and in laboratory techniques.
	Indicates the authorized representative in the European Community.

How to Obtain More Information

Contact Information



Tetracore, Inc.
 77 Upper Rock Circle
 Suite 600
 Rockville, MD 20850-3986
 United States of America

Tel: 240.268.5400
 Fax: 240.268.1107
 E-mail: customersupport@tetracore.com
www.tetracore.com





Emergo Europe
 Westervoortsedijk 60
 6827 AT Arnhem
 The Netherlands


Important Safety Instructions


Always comply with the following precautions to avoid dangerous situations and ensure proper T-COR 8 performance.

The following symbols are found throughout Quick Start Guide:

 **CAUTION** Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or damage to the equipment.

 **WARNING** Indicates a potentially hazardous situation which, if not avoided, may result in serious injury or death.

 **NOTE** Provides information pertinent in using T-COR 8. Please read note carefully before using the product.

 **IMPORTANT!** Provides information necessary for proper T-COR 8 operation.

When reading this Quick Start Guide:

- Read all instructions.
- Keep all instructions.
- Heed all warnings.
- Follow all instructions.

Applicable Software Versions


This guide is applicable to the Heart software releases.

Unpacking and Accessories

To unpack the device remove the packing tape and unfold the sides of the shipping box



With the foam end caps still attached to the device, remove the device from the box. Once removed from the box, pull the left and right foam end caps off of the instrument. Remove the roll up insert from the box.

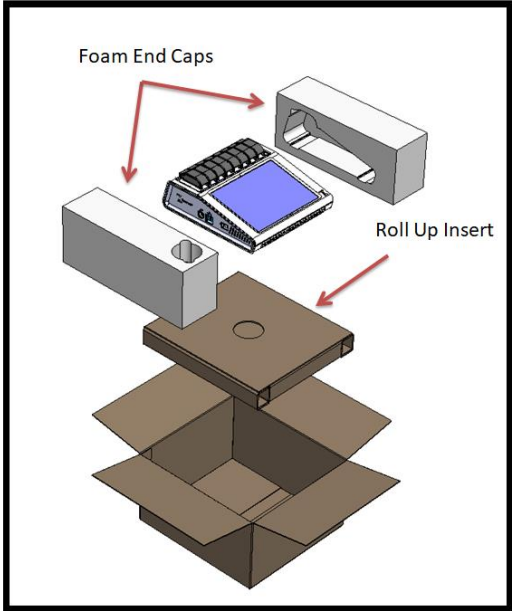

Unfold the large flap on the roll up insert to access the accessories.



Check package for the following items:

- T-COR 8 Device
- AC Adapter
- Power Cord
- Ethernet Cable
- Quick Start Guide
- Hardware Manual
- Software Manual
- LIS Connectivity Specification
- T-COR 8 Tube Rack (optional)
- Magnetic Mixer (optional)
- USB-A to USB-mini (optional)



If the device or any accessories are missing or damaged, contact Tetracore.

⚠ CAUTION USB is only designed for use with Tetracore recommended accessories. Use of any USB accessories other than those recommended by Tetracore could result in damage to the T-COR 8 or the USB accessory.

Required Consumables (not supplied)


The following items are required to use T-COR 8, but are not supplied. To order items, contact Tetracore.


Item	Catalog Number
T-COR 8 CE-IVD Reaction Tube	TC-5029-500

⚠ WARNING User should not substitute required consumables with other options without possible safety and performance consequences.

Set-Up Requirements


Environmental Conditions

 **WARNING: Physical Injury Hazard.** Use this product only as specified in this manual. Using this device in a manner not specified by Tetracore may result in personal injury or damage to the device.

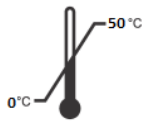

Required working conditions are between 0°C (32°F) and 50°C (122°F).


Ensure the device lies atop a flat, sturdy surface that can sustain at least 7 kg (15 lbs.).


 **CAUTION** Do not obstruct, push objects into, or allow dust to accumulate in the air vents. Leave a minimum of 6 inches (15.24cm) between the vents and any potential obstructions. Do not store device in a low-airflow environment, such as a closed briefcase, while it is running. Restricting the airflow can damage the device and/or cause a fire.





This device is not waterproof or water resistant. Keep T-COR 8 away from liquids to avoid damage to device.

Connecting to Electrical Outlet


 **CAUTION** The AC adapter will only operate using a mains electricity supply within the range of 100-240VAC and 47-63Hz. However, AC power plugs and sockets vary among countries. Using an incompatible power plug or improperly connecting the cable to the power strip or electrical outlet may cause fire or equipment damage.

 **CAUTION** Do not use any type of inverter with this device as it could cause fire or equipment damage.

 **CAUTION** Do not use any type of car charger with this device because it could cause fire or equipment damage.


 **NOTE** Only use a mains power source (e.g. wall power) that can provide a minimum of 220 watts. Most standard mains electricity meets this requirement.


Connect the power cord to the AC adapter before connecting the male DIN connector to the T-COR 8. To connect the AC adapter to the T-COR 8, align the indentation on the male DIN connector with the caret mark on the device and insert fully into the female DIN connector. To safely disconnect power, disconnect the male DIN connector on the AC adapter from the female DIN connector on the device.

 **NOTE** When disconnecting the AC adapter from the T-COR 8, grasp the connector, not the cable itself, and pull firmly but gently without twisting to avoid damaging the cable and/or connector.

Powering the Device

Press the power button to turn the device on and off. The LED ring will illuminate around the button when the device is powered on. Likewise, the LED will not illuminate when device is powered off.

 **NOTE** To avoid loss of data, never turn off the T-COR 8 until all runs display the message “Run Complete”.

 **NOTE** After turning device off, wait at least 15 seconds before powering the device on again.

Magnetic Mixer Accessory

Requires the T-COR 8 Magnetic Mixer accessory, which must be purchased separately.

Connecting to the T-COR 8

If you plan on running any assay that will need the use of the magnetic mixer then you must follow this procedure.

1. Power down the T-COR 8.
2. Identify the USB Cord: Male Type-A to Male Type-B Mini (5-pin) provided with mixer.
3. Connect the Male Type-B Mini (5-pin) end of the USB cable to the magnetic mixer.
4. Connect the Male Type-A end of the USB cable to the T-COR 8.



5. Power on the T-COR 8.

! **NOTE** Magnetic mixer must be connected to the T-COR 8 before the instrument is powered on.

! **NOTE** Magnetic mixer will sound for approximately 20 seconds when powered on.

! **CAUTION** the magnetic mixer is for use with the T-COR 8 only. Use of this mixer with any other machines could cause equipment damage.


Device Utilization

Overview of Device Utilization

There are four ways to use the T-COR 8 device:

1. Stand Alone mode
2. Direct Connection mode
3. Access Point Connection mode
4. Network Connection mode

In Stand Alone mode, the T-COR 8 touchscreen is used – this does not require an external device (for example, a computer). In Direct Connection mode, an Ethernet cable and an external device with the Google Chrome™ web browser installed on any operating system is needed. For Access Point Connection mode, a Wi-Fi capable external device with the Google Chrome™ web browser installed on any operating system is needed. For Network Connection mode, a network of any type (wired or wireless) and an external device with the Google Chrome™ web browser installed on any operating system is needed.

 **NOTE** Internet connection is not required to access the device.


Pre-installed User Accounts

The T-COR 8 has two pre-installed user accounts, *user* and *admin*. The Tech role is assigned to *user*, and the Admin role is assigned to *admin*. When the T-COR 8 is removed from its original packaging, neither *user* nor *admin* require a password. It is possible to change the password. For more information, see *User Administration* in the *Tetracore T-COR 8 Software Manual*.


Stand Alone Mode

In Stand Alone mode, the device does not require an external device (for example, a computer) to set-up or view data. Stand Alone mode allows for device portability. The T-COR 8 has a touchscreen that utilizes an on-screen keyboard when needed. In Stand Alone mode, the idle screen timer is set to dim the screen automatically after 30 seconds (only when running off battery) which can be reversed by simply tapping the touchscreen. All software features are accessible in standalone mode with the exception of the **Reports, Temperature Graph, Touchscreen Calibration, License Page, and Direct Zip Download** which can only be accessed via Google Chrome™ on an external device.

1. Set up the device on a dry, flat area.

 **CAUTION** Do not obstruct within 6 inches (15.24cm), push objects into, or allow dust to accumulate in the air vents. Do not store the device in a low-airflow environment, such as a closed briefcase, while it is running. Restricting the airflow can damage the device and/or cause a fire.

2. Turn on the device by pressing the power switch located on the right side panel. When the LED ring illuminates, the device is ON.

 **NOTE** If the light does not turn on, then the battery has been fully discharged. In this case, do the following:

1. Press power switch again to turn off device. Ensure the power switch button is flush with the outer ring.
 2. Connect the power cord to the AC adapter before connecting the male DIN connector to the T-COR 8.
 3. Align the indentation on the male DIN connector of the AC adapter with the caret mark on the device and insert DIN fully into the female DIN connector.
 4. Wait 15 seconds before powering on the device again.
 5. Press power switch again to turn on device.
-


3. Device is ready for use.

Direct Connection Mode


Direct Connection mode allows the user to access the T-COR 8 software from a computer via Ethernet cable instead of using the touchscreen on the device. It is required that the computer

has Google Chrome™ installed in order to access the T-COR 8 software. All software features are accessible when using Direct Connection mode. To access the software in Direct Connection mode:


1. Set up the device on a dry, flat area.

 **CAUTION** Do not obstruct, push objects into, or allow dust to accumulate in the air vents. Leave a minimum of 6 inches (15.24cm) between the vents and any potential obstructions. Do not store the device in a low-airflow environment, such as a closed briefcase, while it is running. Restricting the airflow can damage the device and/or cause a fire.

2. Connect one end of the Ethernet cable into an Ethernet port on a desktop or laptop computer. Connect the other end of the Ethernet cable into the Ethernet port located on the left side panel.
3. Turn on device by pushing the power switch located on the right side panel. When the LED ring illuminates, the device is ON.

 **NOTE** If the light does not turn on, then the battery has been fully discharged. In this case, do the following:

1. Press power switch again to turn off device. Ensure the power switch button is flushed with outer ring.
 2. Connect the power cord to the AC adapter before connecting the male DIN connector to the T-COR 8.
 3. Align the indentation on the male DIN connector of the AC adapter with the caret mark on the device and insert DIN fully into the female DIN connector.
 4. Wait 15 seconds before powering on the device again.
 5. Press power switch again to turn on device.
-
4. On the touchscreen, log in and go to Menu -> Wifi and Networks -> Ethernet Settings, and ensure the Direct Connect Mode is “On”. Wait until the device’s IP address appears in the bottom right of the touchscreen.
 5. Using the connected computer, open the Google Chrome™ web browser.
 6. Enter the IP address of the device into the URL box of the browser.


 **NOTE** When directly connected to a computer via Ethernet cable, the IP address will always be **192.168.8.2**.

7. Device is ready to use.


Access Point Connection Mode

Access Point Mode allows the user to access the T-COR 8 software from a Wi-Fi client device, such as a phone or laptop. It is required that the client device has Google Chrome™ installed in order to access the T-COR 8 software. All software features are accessible when using Access Point mode. To access the software in Access Point mode:

1. Set up the device on a dry, flat area.


 **CAUTION** Do not obstruct, push objects into, or allow dust to accumulate in the air vents. Leave a minimum of 6 inches (15.24cm) between the vents and any potential obstructions. Do not store the device in a low-airflow environment, such as a closed briefcase, while it is running. Restricting the airflow can damage the device and/or cause a fire.

2. Turn on device by pushing the power switch located on the right side panel. When the LED ring illuminates, the device is ON.

 **NOTE** If the light does not turn on, then the battery has been fully discharged. In this case, do the following:

1. Press power switch again to turn off device. Ensure the power switch button is flushed with outer ring.
 2. Connect the power cord to the AC adapter before connecting the male DIN connector to the T-COR 8.
 3. Align the indentation on the male DIN connector of the AC adapter with the caret mark on the device and insert DIN fully into the female DIN connector.
 4. Wait 15 seconds before powering on the device again.
 5. Press power switch again to turn on device.
-

3. On the touchscreen, log in and go to Menu -> Wifi and Networks -> Switch to AP Mode
 4. Using the client device, connect to the new open Wi-Fi network. The SSID is the device's serial number. Wi-Fi security is not an option at this time. Open the Google Chrome™ web browser on the client device.
 5. Enter the wireless IP address of the T-COR 8 into the URL box of the browser.
-


 **NOTE** When directly connected to a computer via Access Point mode, the IP address will always be **192.168.8.3**.

6. Device is ready to use.

Network Connection Mode

In Network Connection mode, the T-COR 8 allows for remote access, control and data analysis via a wired or wireless network. An external device with Google Chrome™ installed must be connected to the same network as the T-COR 8 in order to access the device. After a successful network connection is established, the IP address assigned to the T-COR 8 will be displayed in the lower right corner of the touchscreen. Open the browser and enter the appropriate IP address into the URL box. All software features are accessible when using Network Connection mode.

1. Set up the device on a dry, flat area.


 **CAUTION** Do not obstruct within 6 inches (15.24cm), push objects into, or allow dust to accumulate in the air vents. Do not store the device in a low-airflow environment, such as a closed briefcase, while it is running. Restricting the airflow can damage the device and/or cause a fire.

2. Turn on the device by pushing the power switch located on the right side panel. When the LED ring illuminates, the device is ON.

3. Connecting to network via Ethernet cable or Wi-Fi:

- a. Ethernet Cable: Connect one end of the Ethernet cable into an Internet access point (for example, a modem or wall Ethernet drop). Connect the other end of the Ethernet cable into the Ethernet port located on the left side panel.
- b. Wi-Fi: Press the **[Menu]** button located on the bottom right of the display in the software. Press the **[Wifi and Networks]** button in the window. Press **[Rescan]** to search for available Wi-Fi networks. Select the network to connect to.

4. Using any computer connected to the same network, open the Google Chrome™ web browser.
5. Enter the IP address of the device into the URL box of the browser.


 **NOTE** The IP address of the device can be found in the lower right corner of the touchscreen.

6. Device is ready to use.

A Quick Guide to Run Setup

The instructions below are only a basic overview. For detailed information about each screen and functionality, see *Tetracore T-COR 8 Software Manual*.

Creating a New Run

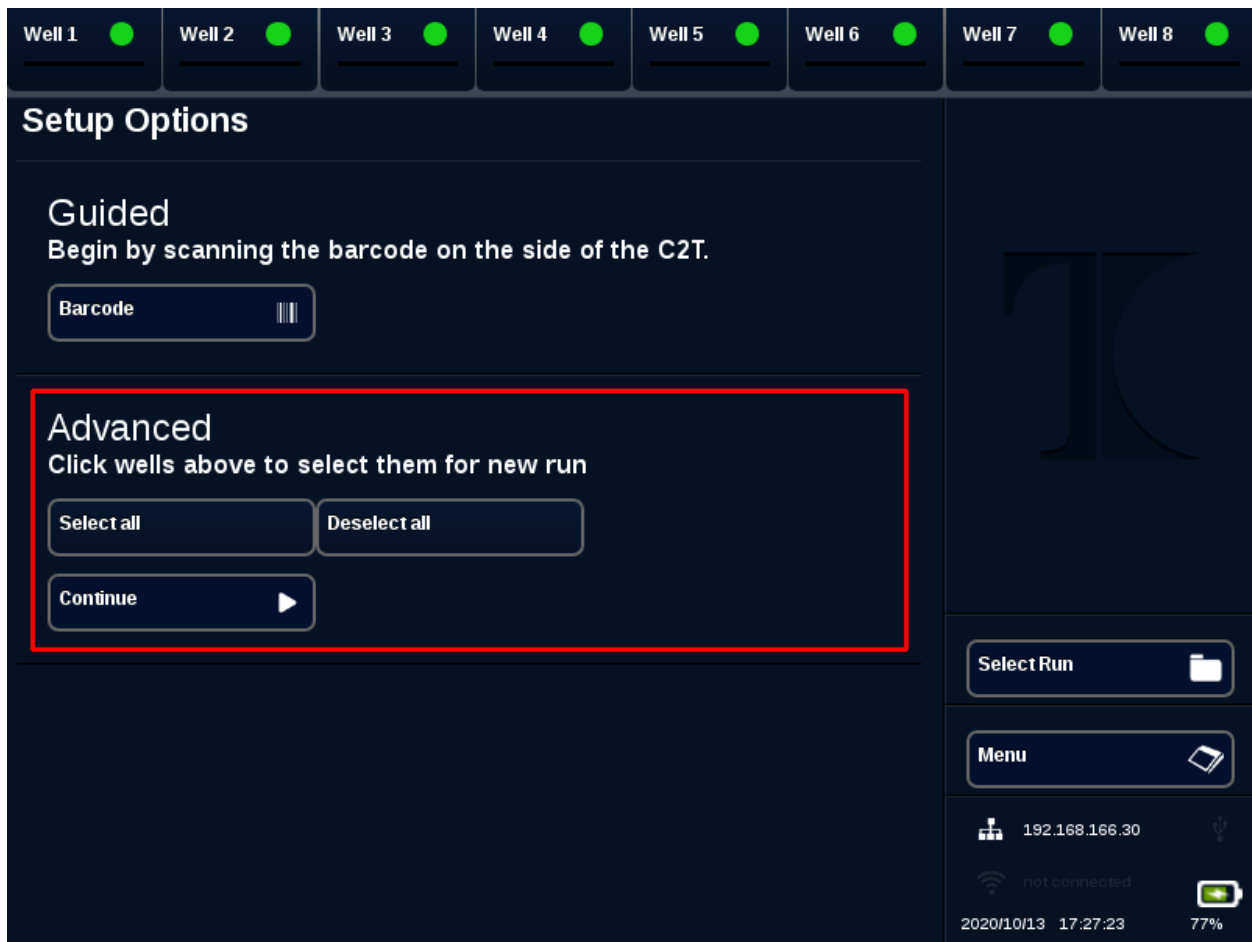

1. Sign in to the T-COR 8
2. Open the **[Menu]** and select **[New Run]**.
3. Select well(s) to be used by highlighting the corresponding well header(s). Highlighted well headers will have a **green circle**. **[Select all]** selects all available wells. **[Deselect all]** deselects all wells.

! **NOTE** When wells are currently running, those wells will be highlighted with a **red bar**. These wells are not available until the current run(s) are completely finished.

4. Select **[Continue]** to proceed onto the **New Run Summary** page.

Well 1 ● **Well 2** ● **Well 3** ● **Well 4** ● **Well 5** ● **Well 6** ● **Well 7** ● **Well 8** ●

Sample 1 (∅) **Sample 2** (∅) **Sample 3** (∅) **Sample 4** (∅) **Sample 5** (∅) **Sample 6** (∅) **Sample 7** (∅) **Sample 8** (∅)

New Run Summary

Click a well above to modify it

Run name

Comments

Apply to all wells

Assay

(∅) ▼ Barcode Show Protocols ⓘ

Start Run ▶

Menu 📄

192.168.166.30


not connected

2020/10/13 17:27:35 77%

5. Enter a **Run Name** (optional). Enter any **Comments** (optional). This can be altered after the run has started, but cannot be altered after the run has completely finished.
6. Choose an assay to be used for this run from the dropdown menu underneath **Assay** to apply to all wells. Each well may also be assigned with different assays (see step 7).

! **NOTE** Assays must already be pre-defined. If an assay is not listed, create an assay before creating a new run (see *Editors*). **This can only be accessed by Admins and Lab Directors.**

7. Should you choose to run more than one assay, select the well with a yellow circle well header.

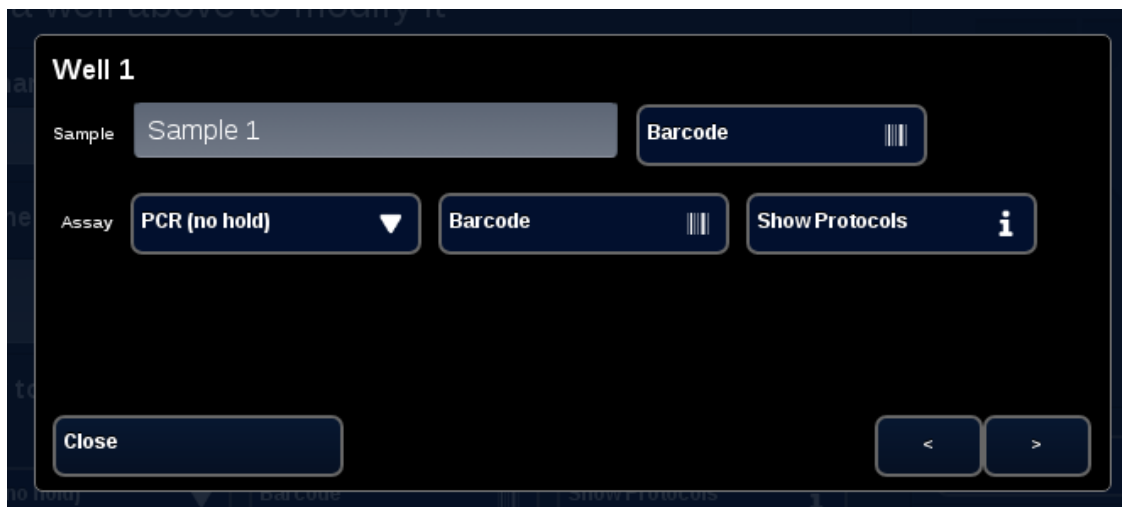

Enter **Sample** name for that well and choose an assay to be used for this well from the dropdown menu next to **Assay**. If available, an assay can also be scanned by pressing the **[Barcode]** button next to the Assay header. Press **[Close]** to save and exit the window.

8. Repeat step 7 for each well to be used in the run.

! **NOTE** If a barcode for the assay is available, it can be scanned into the device using the barcode reader located on the right panel of the device. Press the appropriate **[Barcode]** button to initiate the barcode reader. Hold barcode approximately 3 inches (7.62cm) away from scanner. The barcode scanner will terminate scanning after 10 seconds if no reading is captured.

⚠ **WARNING: Laser Radiation: Class 2 Laser Product.** Do not stare into barcode beam. Never direct a laser at other people, highly reflective surfaces, or any other object not intended for designated use.

9. Press the **[Start Run]** button.

After the run has been started, the software will automatically display a summary table of the samples and their assay(s). Data can also be viewed and analyzed by selecting a particular **[Graph]** channel (**only available for Admins, Directors, and Molecular Directors**). The **Run Name**, **Comments**, and **Sample** information can be altered while the run is in progress under **[Run Info]**. Once the run is completed, user cannot change **Run Name**, **Comments**, or **Sample** information. Previous runs can be viewed while runs are in progress.

Index

A

AC Adapter..... 8, 9, 10, 13, 14, 15
 Access Point Connection.....15
 Accessories7, 8
 Assay(s)..... 11, 18, 19

C

Caution 6, 8, 9, 10, 13, 14, 15, 16
 Contact Information5

D

Direct Connection 12, 13
 Documentation.....4, 5

E

Electrical9
 Environmental Conditions9
 Ethernet Cable 8, 12, 13, 14, 15, 16

G

Graphs13, 19

L

LED..... 10, 13, 14, 16

N

Network.....12, 16

P

Password12
 Power..... 9, 10, 13, 14, 15, 16
 Power Cord8, 13, 14, 15

S

Samples.....4, 19
 Setup.....17
 Stand Alone mode 12, 13
 Symbols.....5, 6

T

Temperature.....5, 13
 Touchscreen 13, 16

U

User Accounts12

W

Warning 6, 8, 9, 19